



Emergency Evacuation Chair Instruction Guide




During emergency evacuations elevators do not always work, in some cases they are purposely shut-off to avoid other dangers.

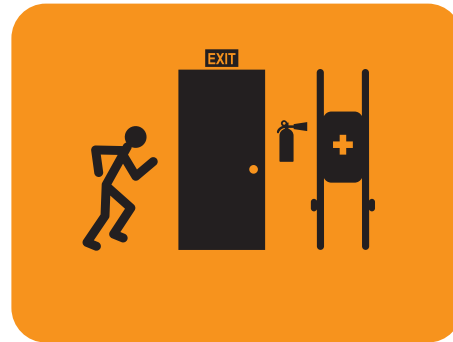

Evacuation chairs should be easily accessible:
Mounted on wall near stairwells.

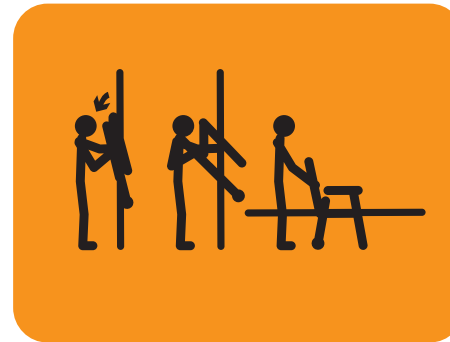

Remove from wall and unfold.
Secure latches. Remove front handles for better access.

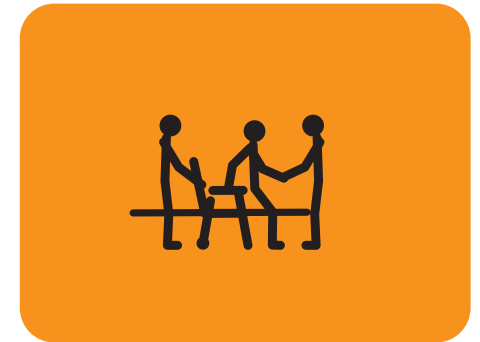


Assist evacuee into chair.
Replace handle (if removed).
Secure chest harness and arm/leg straps as needed. Wheel evacuee to stairwell for evacuation.



With two people lift chair using handles. Keep arms extended and relaxed. The additional side handles allow more people to assist. Side handles may also be removed for narrow stairways.



Evacuate downstairs with evacuee orientated as shown above.

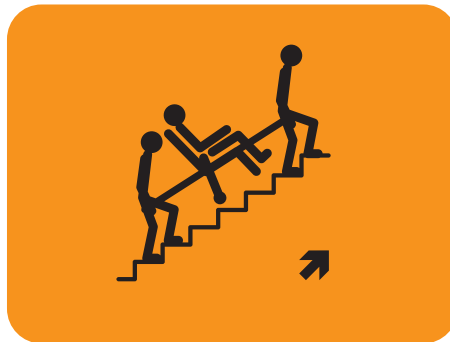

Relocate upstairs with evacuee orientated as shown above.



Once out of the building, evacuee can be quickly wheeled to safety.